

# LiteShower™ Wheelchair-accessible Portable Shower Stall

Setup in minutes, take showers anywhere at home

## How It Works

1. Hand-held shower head with 7' hose connects to indoor standard faucet (quick-connect adaptor included);
2. Shower tray sits on floor. Tray has flexible walls that bend flat under wheelchairs but bounce back afterwards;
3. (2) free-standing shower screens snap together around the tray. Shower curtains open and close on all four sides;
4. Waste water in the tray is pumped to sink. Pump sits outside the tray and has suction hose going into the tray.



## Products

System	Overall Size	Folded Screen Size	Weight
Standard Model – Complete	3' wide x 4' long x 4' tall, bi-fold screens	4' long x 4' tall x 2" thick each	~ 20 lbs
Recliner Model – Complete	3' wide x 7' long x 3' tall, tri-fold screens	4' long x 3' tall x 3" thick each	~ 25 lbs
Standard Model – Less Pump *	3' wide x 4' long x 4' tall, bi-fold screens	4' long x 4' tall x 4" thick each	~ 15 lbs
Recliner Model – Less Pump *	3' wide x 7' long x 3' tall, tri-fold screens	4' long x 3' tall x 3" thick each	~ 20 lbs

\* Customers use their own water pumps for pumpless models.

A complete set includes:

- (1) handheld shower head with 7' hose and on/off control;
- (1) adaptor to connect hose to faucet;
- (1) shower divert valve to connect hose to existing shower;
- (1) self-support, no setup, wheelchair-accessible shower tray;
- (1) pair of folding screens with sliding curtains and hooks;
- (1) pump with hose, filter to dispose waster water into sink;

Assembly Requirement:

- 1. Replace threaded aerator with faucet adaptor (one-time)
- 2. Assemble screens with a screwdriver (one-time)
- 3. Hand-tight inline filter, fitting, and hose to pump (one-time)



1-800-561-3576

[www.LiteShower.com](http://www.LiteShower.com)

Nova Health Products, 13144 Midlothian Turnpike # 1E, Midlothian, VA 23113  
All rights reserved | Wheelchair-accessible Portable Shower | Hazmat Shower

Made in USA  
Patent-pending

